



LiquiBand® Octyl 0.4 Topical Skin Adhesive

LiquiBand® Octyl 0.4 Topical Skin Adhesive is a high strength 2-octyl cyanoacrylate formula designed to provide safe, effective topical wound closure.

					
MIKROBIELLE BARRIERE	HOHE VISKOSITÄT	STARK & SICHER	EINZIGARTIGE SPITZE	HERGESTELLT IM VEREINIGTEN KÖNIGREICH	LAGERUNG BEI ZIMMERTEMPERATUR

✓ 2-OCTYL-CYANOACRYLATE FORMULA

- ▶ Durable and flexible coverage
- ▶ High viscosity
- ▶ Competitive wound closure strength when compared to market leading devices

✓ MICROBIAL BARRIER PROTECTION

- ▶ Effective barrier to gram positive, gram negative and fungal microbes



✓ EASY TO USE APPLICATOR

- ▶ Ready to use out of the box
- ▶ Contoured tip for controlled delivery of adhesive
- ▶ Transparent for visualization

✓ HIGH PATIENT SATISFACTION

- ▶ Low exothermic reaction
- ▶ Atraumatic wound closure



Application Technique


LiquiBand® Octyl 0.4 Topical Skin Adhesive

- ▶ Ensure that the wound is thoroughly clean and dry and wound edges appose readily.



- ▶ While holding upright, squeeze middle of the LiquiBand Octyl 0.4 device until a cracking sound is heard.

- ▶ Invert the tip and gently squeeze up to three times to prime the device. When the tip turns purple, LiquiBand Octyl 0.4 is ready to use. Ensure that the wound is thoroughly clean and dry prior to closure.

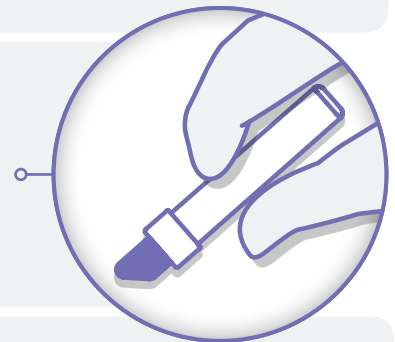

- ▶ While holding the wound edges together, apply a single, even coat of LiquiBand Octyl 0.4 over the length of the wound, using the tip of the device to distribute and prevent pooling of adhesive.
- ▶ Continue to hold the wound edges together for approximately 60 seconds.

LiquiBand® Octyl 0.4 Topical Skin Adhesive

Description	Product Code	Volume/Ea	Ea/Box
LiquiBand® Octyl 0.4	72014001	0.4 g	6/box



T 1-855-458-3777
 E info@liquiband.com
 W liquiband.com